YOUR DISPOSABLE EXCELLENCE

DISPOGEL (14x18 – 14x24 – 11x26) cm

IT: SACCO GEL CALDO/FREDDO

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EN: GEL PACK HOT/COLD

COLD INDICATIONS

Haematomas, sprains, injuries, hyperthermia and whenever cold application is required.

HOT INDICATIONS:

Lumbago, stiff-neck, rheumatism, hypothermia and whenever hot application is required.

INSTRUCTIONS – FREEZER:

Put the gelatine bag in the freezer for at least 1.30/2 H. Take it out, put it into its pocket or wrap it up in a cloth and apply where required.

INSTRUCTIONS – HOT WATER

Immerse the gelatine bag in the boiling water for max 5 minutes. Take it out, wipe it, put it into its pocket or wrap it up in a cloth and apply where required.

INSTRUCTIONS – MICROWAVE OVEN

Put the pack in the microwave for max. 1 minute at 650 Watt. Remove and shake it gently to distribute the heat uniformly. Place the pack in the specific pocket or cloth and apply where required

CAUTION

Do not swallow the contents, do not break the bag. Keep out of children's reach. Put the gelatine bag always in its pocket or wrap it up in a cloth to avoid burns. Before using on babies, children, aged people and people affected by diabetes, cardiovascular and circulation disorders or skin insensitivity (syndrome of Raynaud) consult a doctor. If in doubt on the treatment to carry out or if the symptomatology persists, consult your doctor. Do not apply on injured skin. For external use only. Do not reuse if the bag is damaged (e.g. holes, cuts).

In the event of any serious safety incident occurring by using the medical device, please report it to the Manufacturer and to the Competent Authority of your Country.

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Dispotech Srl